FR

PA0010115

WINNOCARE

by

ASKLÉ SANTÉ

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Matelas en mousse Viscoélastique

CE

CONSEILS

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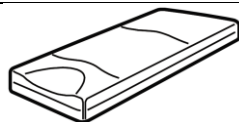
## User Manual Viscoelastic foam mattress

### RECOMMENDATIONS

The main way to prevent bedsores when in bed is to change the pressure points of the body on the mattress on average every three hours to limit a too lengthy compression of the vessels opposite bone protuberances and to allow tissue irrigation. Do not tuck the bedlinen in too tightly to maintain the efficiency of the mattress.

### PRINCIPLE

The Alova mattress is a static mattress for the prevention and treatment of bedsores.

The low resilience Alova viscoelastic foam provides 3 different supports that reduce the contact pressure of the body on the support. A procline integrated into the surface of the Alova mattress on the part in contact with the lower limbs makes it possible to relieve the heels. The combination of these principles improves blood circulation and minimises the pressure on bedsores prone areas.

### INSTRUCTIONS

For patients with reduced mobility and "Average to High" bedsores risk (according to the Braden scale or any other valid scale and depending on the physician's opinion). Helps to treat stage 1 to 2 bedsores according to EPUAP.

### RECOMMENDATIONS FOR USE AND COUNTER INDICATIONS

There are no specific counter indications for the use and place of use. The mattress is designed to be installed on a medical bed frame at the location designed to this effect. The mattress must be used with the delivered cover.

In the case of the use of a bed with side barriers, the minimum distance between the top of the mattress and the top of the barriers must be at least 22 cm. In the case of use of a bed with several functions, check the correct distribution of the support pressure.

### TECHNICAL DATA

Products						
References	VMA62 VSA6ST+ VMS	VMA63 VSA7ST + VMS	VMA64	VMA70 VMA80	VMA117XL ou VMA117XL/RC VMA137XL ou VMA137XL/RC	VMA120BAR VMA140BAR
Sizes	196x87x14 cm	196x87x14 cm	196x87x14 cm	196x70x14 cm 196x80x14 cm	195x117x17 cm 195x137x17 cm	196x117x17 cm 196x137x17 cm
Spec Foam Sup	PU VE : 80kg/m³ 1.8 kPa	PU VE : 80kg/m³ 1.8 kPa	PU VE : 80kg/m³ 1.8 kPa	PU VE : 80kg/m³ 1.8 kPa	PU VE : 85 kg/m³ 1.7 kPa	PU VE : 82kg/m³ 2.2kPa
Spec Foam Inf	PU : 18kg/m³ 3.0 kPa	PU : 28kg/m³ 3.4 kPa	PU : ≥34 Kg/m	PU : ≥34 Kg/m³	PU : 28 kg/m³ 3.6 kPa	PU : 41kg/m³ 4.5 kPa
Min and max patient weight	30 to 120 kg	30 to 150 kg	30 to 180 kg	30 to 135 kg	30 to 150 kg	135 to 270 kg
Materials used for the foam mattress	100% PU					
Materials used for the cover	Knit PES (except CIC with PA Knit) - coating PU					

### MAINTENANCE/DISINFECTION

Cover						
	Moderate wash at 90°C	Maximum authorised chlorine concentration 5000ppm	Do not iron	Do not dry clean	Drying with reduced thermal constraints	Use of an authorised surface detergent/disinfectant product

The delivered removable cover is used to protect the mattress from all contamination by effluents. If the cover is deteriorated it is imperative to change it and not to use the mattress without the cover. The cover can be washed and disinfected in a washing machine at 90°C. The cover can be disinfected using current EC valid disinfectants (do not use corrosive products) The cover must not be ironed or tumble dried. The purpose of the cover is to limit the risk of cross contaminations.

Item treated with a biocidal substance that is safe for the user. [www.winnicare.fr](http://www.winnicare.fr)

### PACKAGING AND INSTALLATION

The mattress is delivered with its zipped removable cover. The product is ready for immediate use. Place the mattress by orienting the label sewn to the cover at the head or in compliance with the symbols on the top surface of the cover. The sheet must not be fixed to the mattress in order not to cancel its effects.

The mattress must be installed with the viscoelastic material on the top.

The top part is marked and easily identifiable on the mattress. The cover is also marked. The heel relief part is indicated by a symbol representing feet. (not the case for all covers)

### WARNING

(Re)assess the risk of patients being trapped in the non-moving parts of the medical bed associated with the "therapeutic mattress and accessories and the articulated bed base positions". The use of segmented or full-length side barriers must be required.

### PRECAUTIONS FOR USE

Avoid prolonged damp contacts with the foam.

Systematically protect the mattresses using rigorously dried delivered protection.

Prohibit all scouring, stripping products or solvents and "piercing-sharp" objects in direct contact with the cover.

### REUSE

Before reusing for another person, clean and disinfect the mattress with a "detergent/disinfectant" product solution.

We recommend the replacement of the cover.

### MAINTENANCE

Regularly carry out a visual inspection of the condition of the foam: the presence of a visible collapse of the ALOVA material and an uneven and very slow return to shape of the foam are the signs of wear criteria that compromise the properties of the support in helping to prevent bedsores. Inspect the condition of the cover yearly to check the condition of the foam, the surface aspect of the cover and the closures (zippers). In order to check for the absence of holes, tears or problems with the zippers, examine the outer surface in natural daylight.

### CONDITIONS FOR USE AND STORAGE

The pneumatic supports with telescopic cells should preferably be stored flat, away from direct light and excessive humidity.

	Use	Storage
Temperature range	15°C – 45°C	-5°C – 60°C
Humidity range	30% - 70%	10% - 90%

### ELIMINATION OF MEDICAL WASTE

Do not dispose of the product outside dedicated areas.

### WARRANTY

The warranty period for ALOVA mattresses is 3 years. The warranty begins on the purchase date from the distributor. In the case of a single patient use, the warranty covers the support and its protection. In the case of multi patient use, the warranty period for the protection is reduced to 1 year.

This warranty does not cover normal wear and tear and is not a substitution of legal guarantees. Please contact your distributor and present the incriminating product. The distributor will carry out the necessary procedure with our company in order to either repair or replace the product.

### PRODUCT LIFE SPAN

The estimated life span of ALOVA mattresses is 6 years.

### SUITABILITY FOR USE:

(max) Pressure in mmHg before ageing, using the maximum validated weight, measured flat (0°)

Reference	Max. weight User	Head	Buttocks	Heels
VMA Mattress + Promust PU cover	150 kg	89	60	75
BAR Mattress + Promust PU cover	270 kg	130	85	93

### Composition of the parcel

1 mattress  
1 cover  
1 set of instructions for use

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