



DOOR CONNECTOR

TRANSITION GATE. THROUGH THE DOOR - QUICK AND EASY



Pull the steel wire cord to activate and unlock the door connector.
See step-by-step guide on next page.

Door connectors are used for patient transfer from room to room. WinnCare's simple and reliable mechanism is used for transferring patients from room to room, e.g. bedroom to bathroom, thereby reducing the number of total lifts required. Fewer lifts creates a better working environment for the carer and safer transfer for the patient.

WinnCare's door connector, as a wall mounted solution, is the preferred choice by architects and building contractors, because of its elegant and compact design.

The door connector is suitable for most wall widths. And because it is mounted very close to the wall surface taking up little space in the room, the desired room covering area is exploited optimally.

WinnCare's door connector is only 300 mm wide and does not require costly changes to the building's electrical system, as it operated manually.

ITEM NUMBERS

Version	Item no.
Traverse track 135 mm - Connector 135 mm - Traverse track 135 mm	016-01300
Fixed track 135 mm - Traverse track 135 mm	016-01304
Fixed track 75 mm - Traverse track 135 mm	016-01302

PRODUCT HIGHLIGHTS

- Only 300 mm wide
- Flexible for fixed and traverse systems
- Manual operation – no costly electrical installations required
- Traverse is locked and unlocked by simply pulling the cord
- Several models to suit individual configurations
- Fail safe mechanical features



DOOR CONNECTOR: STEP-BY-STEP GUIDE



1. Pull the steel wire cord to activate the door connector.



2. Slide the traverse towards the door connector.




3. When the traverse is aligned with the door connector, it will automatically lock into position with a click sound ready for room to room transfer.




4. Once the ceiling lift has been moved into the other room, pull on the cord to unlock the mechanism and to move the traverse.

TECHNICAL SPECIFICATIONS



	Connector 135mm	Fixed track 75mm
Length	300 mm	220 mm
Height	145 mm	145 mm
Width	100 mm	100 mm

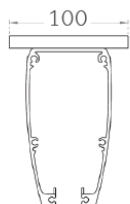

Drawings refer to connector 135 mm

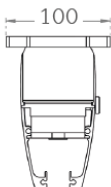


Drawings refer to fixed track 75 mm

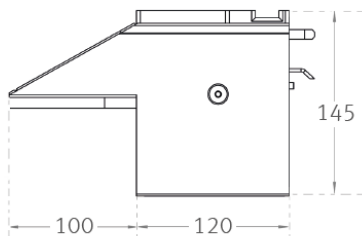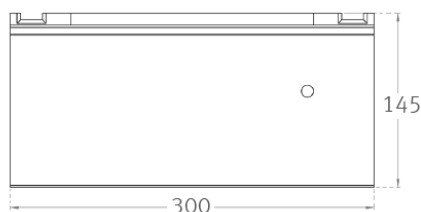

Traverse track 135 mm
Connector 135 mm
Traverse track 135 mm



Fixed 135 mm
Traverse track 135 mm

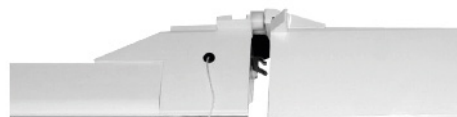

Fixed track 75 mm
Traverse track 135 mm

Consult the Technical manual for further specifications.
All figures and measurements are for guidance only and may differ in actual situations.
WinnCare reserves the right to changes without notice.
Product photos may include accessories.

CE MDD93/42/EEC
Class I

FOR FURTHER INFORMATION, PLEASE CONTACT:

winnCare.com
E-mail: info@winnCare.dk
Telefon: +45 70273720

WIN CARE